


Using the Sling

1. Unfold the sling and place on bed as illustrated in photo. Seatbelts and supports should be checked for any wear, knots or tangles. Make sure that the links on the chain supports are closed and tightened.

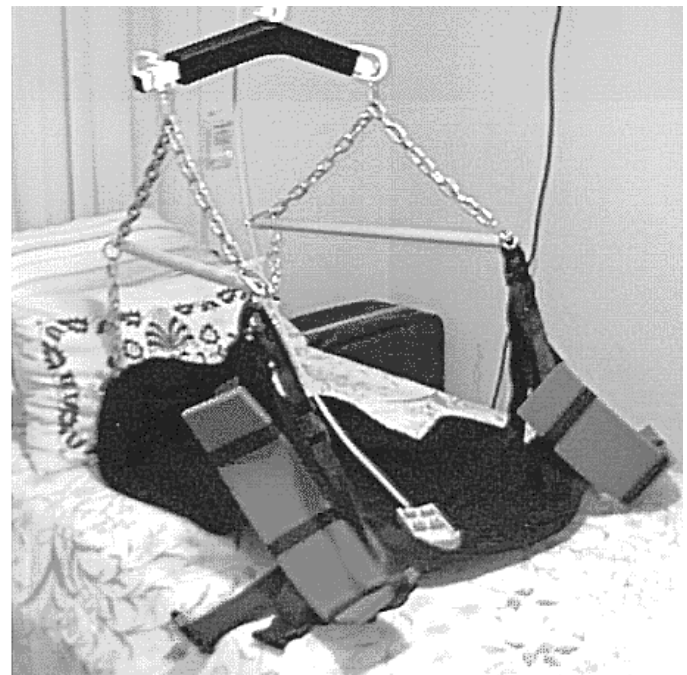

2. Place a small towel at the bottom (foot) end of sling. Position person on sling with hips fully on lower section, groin should extend just slightly over the bottom end of the sling. Place small pillow under head if desired. Do up seatbelts beginning with the bottom one first. Note: DO NOT USE THE WOODEN SPREADERS FOR SUPPORT. THESE ARE DESIGNED FOR KEEPING THE CHAINS SPREAD ONLY.



3. Lower hoist, hook support links over hoist ends as shown in photo and raise hoist until the sling is just about to lift from bed. Unsnap blue leg supports and place under thighs then reconnect the snaps. Once both are done up, adjust for position if necessary.

4. Raise hoist until sling is just off bed. Check to make sure all links and buckles are tight and that everything is properly supported. Once this check is done, sling may be positioned as necessary.

5. To get out of sling, drop slowly onto bed. Lower hoist to loosen chain supports then remove one at a time, dropping over the side of the bed. Remove leg supports. To store fold chain supports and leg supports into center then fold sling in half and fold in end. Sling may be cleaned after removing chain supports. Use mild detergent, warm water. Hand wash, rinse and hang to dry.